

VisAtten.01 Scanner RunSheet

Date _____ Time (hosp) _____
 Time (scan start) _____ Time (scan end) _____
 Subject # _____ Experimenter _____

Version:			
1	2	3	4

<input checked="" type="checkbox"/> fMRI Set-up
MRI Screening Forms
Consent Forms signed & reviewed
Restroom for Subject
Pregnancy Test (for women)
MRI Instruct: no movement, no metal
Experiment Instructions
Set monitor resolution to 800x600 for Cigal
Video Switcher (first eye tracker, then left monitor)
Eye-tracker set-up
Glasses set-up (if needed)
Check output
Auto Scale Cigal
Check Response Buttons (wordpad)

Resting State	<input checked="" type="checkbox"/>	Notes: Prep dual screen (PC dual mode & in CIGAL). Hit "w" <i>after</i> scanner button turns green. Eyes open!
4:03 Resting 1		
4:03 Resting 2		

FSPGR	<input checked="" type="checkbox"/>	Notes: Eyes closed!
4:53 FSPGR 3D		
<i>Ask for the SCIC image (Clariview post-processing for head coil intensity correction)</i>		

T2 Flair		✓	Notes: Run "Structural Practice During scan. Check output using macro
5:22	T2 Flair/ Struct Prac		
<hr/>			
Calibrat ion		✓	Notes: Change Video Switch to eye-tracker monitor. Run Calibration in "ViewTracker" Software. Remind participant of experiment instructions. Record Eyes once scan starts.
-	-		
<hr/>			
Exp Runs		✓	Notes: Remind subject of instructions.
6:18	Func Run 1		
6:18	Func Run 2		
6:18	Func Run 3		
6:18	Func Run 4		
6:18	Func Run 5		
<hr/>			
PD/T2		✓	Notes: Eyes closed
4:36	PD/T2		
<hr/>			
DTI		✓	Notes: Eyes closed
4:48	DTI 1		
4:48	DTI 2		
<hr/>			
High- Res Resting State		✓	Notes: Eyes open!
~7	High-Res Resting		

Notes:

 Copy e-prime data Copy eye-tracking files Copy physio data